



Cleaning Recommendations for micrAgard™ PLUS Wipe Clean Fabric

Thanks to the intelligent design of micrAgard™, the fabric maintains its antibacterial and antimicrobial properties even if it is never cleaned. However, to keep it looking in the best condition possible or to adhere to your organisation's infection control guidelines we have outlined some further recommendations below.

- For removing accumulated dust and grit, use a regular vacuum cleaner or wipe with a damp cloth.
- For more stubborn dirt, we recommend upholstery cleaner. Please use according to manufacturer's instructions.
- Any stains that the upholstery cleaner won't remove – such as spots of tar – try removing with white spirit. Dab it on with a piece of absorbent cloth or cotton and then wipe it off. Alternatively, try using washing up liquid on a damp cloth.
- Once washed, allow the bag to dry naturally. Do not tumble dry.
- Certain chlorine and alcohol based products Clinell Clorox wipes, Actichlor tablets, Steri-7 wipes and isoprenol spray are also suitable for more stringent disinfection if required e.g. in a medical environment.

IMPORTANT: Always wipe surfaces again with water after using any cleaning agent, ensuring no residue is left on the material.

We do not advise using products or methods not listed on this document and we cannot be held responsible for damage caused to bags on which unlisted products or methods are used.

